


Proper Procedure on Turning ON/OFF of the Miri TL

Introduction

It is important to properly turn ON/OFF the Miri TL incubator and the Viewer Software in order to avoid malfunctions. Please, closely follow the procedure below.

Turning On

1. Ensure that the Miri TL, the Viewer software and the router are all properly plugged to an electrical outlet. Instances like when the router is not connected, there will be no communication between the incubator and the Viewer Software.

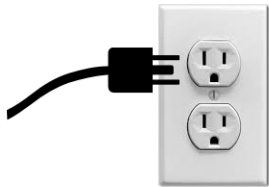

2. Turn on the Viewer Software (PC) and the router.



3. Turn on the Miri® TL incubator by pressing the rocker switch button at the back, right side.



4. Turn on the tablet on the Miri® TL incubator by pressing “TL on/off” button.



Shutting Down

1. Turn off the tablet on the Miri® TL incubator by pressing “TL on/off” button. Please wait for the tablet to turn off.



2. Turn off the Miri® TL incubator by pressing the rocker switch button at the back, right side.



3. Turn OFF the Viewer Software (PC) and the router.

